



## **EpiPin and Microcurrent**

### **1. Cleanse**

Any gentle cleanser from your favorite skincare line works

### **2. Exfoliate**

Use your favorite mild exfoliator (scrubs, enzymes, mild AHA's)

### **3. Microcurrent**

Perform the Microcurrent steps as recommended by your Microcurrent device provider. Remove all excess conductive Gel before EpiPin.

### **4. EpiPin**

Choose the appropriate EpiPin Tip for your client's skin condition (Hydrate, Brightening, Elasticity)

EpiPin dry skin first, then apply the EpiEssence and EpiPin again.

### **5. Mask**

Apply the EpiMask Hyaluron Sheet Mask onto face and neck. Remove after 15min.

### **6. Finish**

Apply a light, hydrating moisturizer and spf