



EpiPin and Dermaplane

**Combining Dermaplane and EpiPin is a very effective combination!
Please be advised to check with your State Board if Dermaplane falls
within your scope of practice.**

1. Cleanse

Any gentle cleanser from your favorite skincare line works

2. Degrease

Wipe the skin with the Dermaplane Pro AHA/BHA Solution to thoroughly and gently de-grease the skin.

3. Dermaplaning

Using a 10R coated blade follow the DermaPlane PRO dermaplaning technique remembering to keep blade on a 45-degree angle while holding clients skin taught.

4. EpiPin

Choose the appropriate EpiPin Tip for your client's skin condition (Hydrate, Brightening, Elasticity)
EpiPin dry skin first, then apply the EpiEssence and EpiPin again.

5. Mask

Apply the EpiMask Hyaluron Sheet Mask onto face and neck. Remove after 15min.

6. Finish

Apply a light, hydrating moisturizer and spf