



EpiPin and LED

1. Cleanse

Any gentle cleanser from your favorite skincare line works

2. Exfoliate

Use your favorite mild exfoliator (scrubs, enzymes, mild AHA's)

3. EpiPin

Choose the appropriate EpiPin Tip for your client's skin condition (Hydrate, Brightening, Elasticity)

EpiPin dry skin first, then apply the EpiEssence and EpiPin again.

4. Mask

Apply the EpiMask Hyaluron Sheet Mask onto face and neck. Remove after 15min.

5. LED

Expose the skin for 10-15min with the appropriate LED light.

6. Finish

Apply a light, hydrating moisturizer and spf