



Dr. Grandel Hydro Active Facial with EpiPin

1. Cleanse

Cleanse with Micell Cleansing Water to remove all traces of make-up with cotton rounds. Cleanse again with the Cleansing Milk.

2. Exfoliate

Exfoliate with Effect Peel. Scrub in areas without any broken capillaries and leave on for 5minutes. Steam is not necessary but increases softness of skin for extractions if needed.

3. EpiPin

EpiPin on dry skin first, then apply the Hyaluron Moisture Flash Ampoule + EpiEssence and EpiPin again.

4. Mask

Apply the EpiMask Hyaluron Sheet Mask with Golden Sheet facing down or Dr. Grandel's Hydro Active Gel Mask (3minute mask) Remove Sheet Mask after 15min.

5. Finish

Apply the Hydro Active Eye Contour Gel and Hyaluron Refill Cream.

* EpiPin is 'Product Neutral' and does not promote or endorse any specific product line. We haven't necessarily performed the protocol and ask you to rely on your skin care expertise to determine what products are best to use on your clients.