

## CASMARA OCEAN MIRACLE FACIAL with EpiPin

### 1. Step One

Cleanse the skin using PHASE 1 Keratoclean. APPLY the full cleansing tube, avoiding the eye area. Leave on for 3 - 5 minutes. Then, with fingertips moistened in water, apply circular movements to the face for several minutes. Remove with a steamed towel.

### 2. Step Two

Apply PHASE 2 Ionizable Moisturizing Marine Plasma. Apply the full tube, with an active massage for 8 minutes.

### 3. Step Three

Apply PHASE 3 Moisturizing Serum to face and massage until completely absorbed.

### 4. EpiPin

EpiPin on dry skin first, then apply PHASE 4 Firming & Nourishing Serum and EpiEssence and EpiPin again. Massage any excess serum into skin.

### 5. Mask

Empty the contents of the Phase 1 sachet into a bowl then add the contents of Phase 2 sachet. Mix both components with a spatula for approx 30-40 seconds until an even mix is achieved. **NOTE: if not mixed evenly the mask may not set correctly.** Using a spatula, apply the mask evenly over the skin. You may cover eyes and mouth. Mask can remain on skin for 20 minutes. Remove mask in one piece.

### 6. Finish

Apply PHASE 6, Natural Marine Serum Moisturizing. Apply a small amount evenly over the skin and give the remainder of the tube to the client to take home and use.

\* EpiPin is 'Product Neutral' and does not promote or endorse any specific product line. We haven't necessarily performed the protocol and ask you to rely on your skin care expertise to determine what products are best to use on your clients.