

GLYMED DERMASOUND TREATMENT FOR MELASMA/CHLOASMA

1. Cleanse

Dispense 2 pumps **Mega Purifying Cleanser** into dry hands and massage into dry skin for 1-2 minutes. Remove with warm cloths and/or cotton pads saturated with water. Dispense 1 pump **Master Aesthetics Elite Gentle Facial Wash** into wet hands and work into a lather, massage into the skin for 2-3 minutes. Remove with warm, damp cloth.

2. Cavitation with DermaSound.

3. Exfoliation

Dispense a nickel size amount of **Oxygen Deep Pore** or **YOUTHFIRM** or **Age Management Micro Scrub with PC10** or **Anti- Aging Exfoliant Masque** in a glass dish. Using a soft fan brush, apply the peel or scrub to the face, avoiding the eyes and lips. Allow peel to remain on the skin for 3-7 minutes. If using the scrub or exfoliant masque use nice light effleurage movements for a light mechanical exfoliation. Remove with warm, damp cloths.

4. EpiPin

EpiPin on dry skin first, then apply the **Skin Restoring Fulvic Elixir** all over, then EpiEssence on top and EpiPin again. Apply **Derma Pigment Bleaching Fluid** or **Derma Skin Brightener** in sections to entire face.

5. Mask

Apply a quarter size amount with fan brush of **Skin Bliss Hydrating Masque mixed with Ultra Hydro Gel and Vitamin C Serum** the face. Include lip and lip area. Allow to remain on the skin for 10 minutes. Remove with soft gauze only. Massage residual product into the skin.

6. Balance

Layer 1 pump **Diamond Bright Skin Brightener** to entire face and neck.

7. Sono with DermaSound.

8. Eyes

Finish around the eye area with **Age Management Anti-Wrinkle Eye Cream**.

9. Protect

Finish with an application of **Age Management Environmental Protection Gel 15**. Pat in gently until absorbed.

* EpiPin is 'Product Neutral' and does not promote or endorse any specific product line. We haven't necessarily performed the protocol and ask you to rely on your skin care expertise to determine what products are best to use on your clients