

Dr. Grandel Collagen Treatment with EpiPin

1. Cleanse

Cleanse with the **Cleansing Milk**. In case of heavy Make-up Removal, use the **Micell Cleansing Water** with cotton rounds.

2. Exfoliate

Apply **Effect Peeling** to chest, neck and face. Use as scrub in areas without broken capillaries, use its enzymatic function on the rest of the face (leave on for 5min, does not need to be kept moist)

3. EpiPin

EpiPin on dry skin first, then apply 1-3 full droppers of **Pro Collagen Serum** or **Collagen Boost Ampoule** and EpiEssence and EpiPin again.

4. Mask

Apply the **EpiMask Hyaluron Sheet Mask** and the Golden Sheet (golden side facing down). Remove after 15min.

5. Finish

Apply **Pro Collagen Eye** and **Pro Collagen cream**.

* EpiPin is 'Product Neutral' and does not promote or endorse any specific product line. We haven't necessarily performed the protocol and ask you to rely on your skin care expertise to determine what products are best to use on your clients.