



Thalgo Regnerating Facial with EpiPin

1. Cleanse

Start by using the Thalgo Micellar Cleansing Water to remove any make up on skin, then cleanse the face using Thalgo Gentle Cleansing Milk to break down and remove dirt & debris from skin. Remove cleanser with a steamed towel. If needed, perform a second cleanse if the client is wearing heavy makeup or is very oily.

2. Exfoliate

Using Thalgo Gentle Exfoliator. Apply a small amount to palm of hand, and massage over wet face in circular motions. Remove with steamed towel.

3. Tone

Apply Thalgo Beautifying Tonic Lotion on a soft cotton pad all over face to be sure all exfoliant and cleansing residue is removed. Apply Thalgo Prodige des Océans Essence by gently pressing the solution into the skin over entire face.

4. EpiPin

EpiPin on dry skin first, then apply the Ultimate Time Solution Serum all over face & neck as well as eye area & lips. Apply EpiEssence on top and EpiPin again. Massage in any excess serum into the skin.

5. Mask

Apply Thalgo MCeutic Essential Regenerating Mask evenly over face and allow to sit for 10-15 minutes. Remove and discard mask, massage in any remaining serum.

6. Finish

Apply a small amount of Thalgo Ultimate Time Solution Eyes and Lips to eye and lip areas followed by Thalgo Ultimate Time Solution Cream and SPF

* EpiPin is 'Product Neutral' and does not promote or endorse any specific product line. We haven't necessarily performed the protocol and ask you to rely on your skin care expertise to determine what products are best to use on your clients